Art INJECTOR™

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Art INJECTOR™

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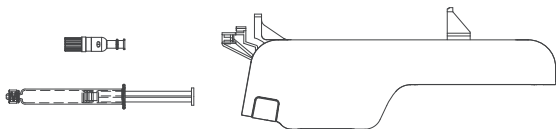
Prepare the main device and pre-filled syringe and cartridge.

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
Turn on the device by pressing the “start” button for 2 seconds.

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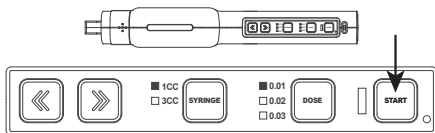
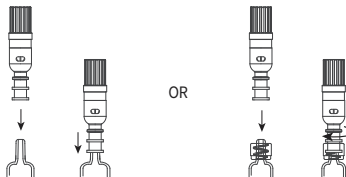
Don't connect too heavily to avoid damaging cartridge.

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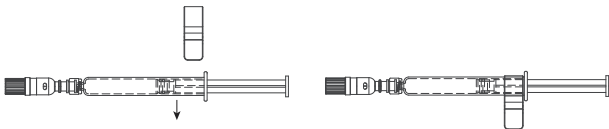
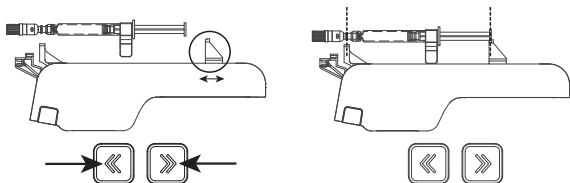
Put square part of cartridge on the bracket and closely place the plunger to the plunger pusher. Also, push the syringe holder on to the device until lock.

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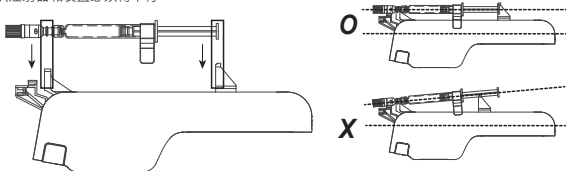
After removing the cartridge cap, remove air from new cartridge by pushing the '<<', '>>' button.

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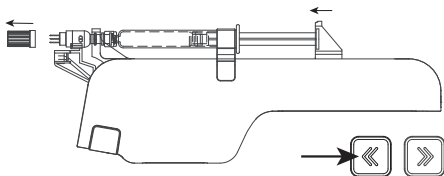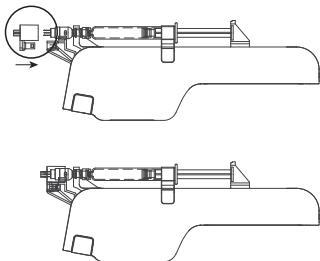
Insert your choice of guide and push forward to the end until cartridge comes out to right length. (You can control the cartridge depth by changing guide)

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Before insert the guide, make sure that cartridge must be fit in the guide hole otherwise it may cause damage.

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
Select syringe size (Base on inner diameter / 1,3cc) and dose volume (0.01, 0.02, 0.03) by pressing button. After, press start button to activate the injection (Green light will be appeared)

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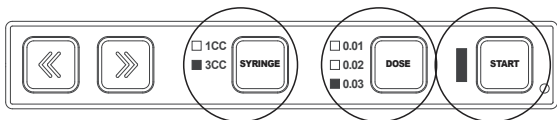
Turn off the injection system by pressing start button (Green light off).
Remove the guide by pulling out.

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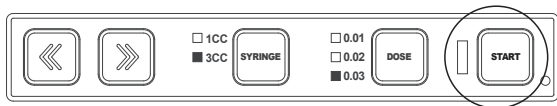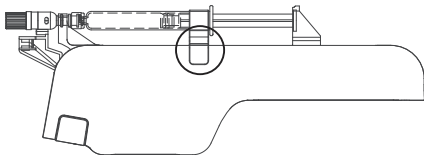
Press bottom part of the holder to remove

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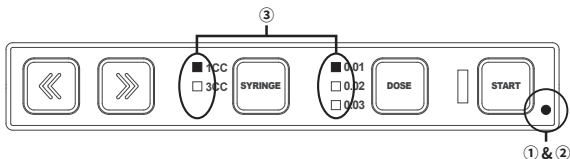
When charging is required : The syringe size lamp and dose volume lamp blink.

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

Push all the way to inject solution and hold until
beep sound finished.

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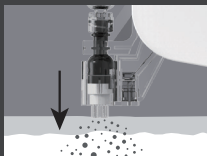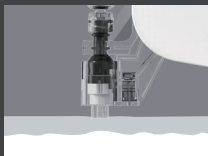Warranty

1. Warranty on this product is from this written warranty card.
2. For more details, refer to the compensation criteria for consumers.
3. This is followed by company policy only on the replaced product.
4. No responsibility of repair, replacement, and refund until the defective product was returned.
5. Original product is only for A/S, but not for the imitation. (Cautious on the imitation)

Guidelines on Compensation for Consumer Damage

	Type of damage		Within warranty	After the warranty
			A/S	A/S
Failure in performance or functions occurring naturally during normal use	When a major repair is required within ten days from the date of purchase.		Product will be replaced Or refund	
	When a major repair on components is required within one month from the date of purchase.		Product will be replaced	
	When a major repair is required within one month from the exchanged product.		Product will be replaced Or refund	
	Impossible to exchange.		Refund	
	Repairable	Defect	Free service	Pay for the repair or Replaced after pay
		Up to three mechanical failures resulting from the same defect, at the 3th failure arising from the same defect	Product will be replaced Or refund	
	Up to various mechanical failures resulting from the same defect, at the 5th failure arising from the same defect			
	Impossible to repair	In case that the company lost the product consumer requests to repair	Product will be replaced Or refund	To be refund by calculating depreciation with 10% increase (Max: Purchasing Cost)
		In case that there are no spare parts in the company within retention period		To be replaced or Replaced after calculating depreciation
		In case that there are spare parts, but in case of impossible of repair		
1) In case of functional break down caused by consumer's abuse - Consumer's mishandling failure (fall, shock, breakage, hard movement) or damage by improper use) - Breakdown or defect by consumer's fault or on purpose - Breakdown or defect from repairing or remodeling by consumer or unauthorized service center - Breakdown or defect from using unauthorized spare parts, consumables, options, etc. 2) Others - In case of defect by natural disaster (fire, immersion in water, earthquake, etc.) - n case of consumable's expire - In case of outside cause.			Pay for the repair	Pay for the repair
※ Components and consumables except for the main body are provided by other warranty period				

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Rating

Input - AC 100~240V 50/60Hz, Output - DC 5V

Please refer to the instruction manual

#1008 Sunil Technopia Bldg, 555 Dunchon-daero, Jungwon-gu,
Seongnam-si, Gyeonggi-do, South Korea.

Tel : +82-2-555-1536 | Fax : +82-2-3142-7670

Service center : +82-70-4680-5612 | E-mail : bestgcs@gcsgroup.co.kr

MADE IN KOREA